

# MAGIC CURVES

## INSTRUCTION MANUAL



Dear Customer:

Thanks for your choosing our products, for the best capability and safety, please read the manual before operation.

Please reserve the manual for your reference.

Copyright ©2020. Manufacturer's User Instruction Manual

# CONTENTS

---

1. Accessories Connection	3
2. Operation Instruction	5
1.Homepage operation instruction	5
2.Magic curve operation instruction	6
3. Anti-Cellulite Panel instruction	7
3. Operation Instruction	8
1. Home Page Operation	8
2.Magic Curve Operation instruction	9
3. Anti-Cellulite operation instruction	10
4. Solution with simple Malfunction	11
5. Specification	11
6. Packing list	12
7. Warm Tips:	13
8. Operation Cautions	14

# Accessories Connection

## 1. Magic Curve Handle connection

■1■ Magic curve handle and Magic curve heads



■2■ Align the handle to the Magic curve head



■3■ Fix the side face plastic clips



■4■ Open the two sides bottom of the plastic clips if need to change the Magic curve head.



## 2. Anti-cellulite handle connection

Temperature Adjustment

Frequency Inten  
sity Adjustment




The Anti-Cullulite Handle no need teardown, After adjust the parameter , Use the Jel while using the handle

### 3. Massage Cloth(Use it while operating the Magic Curve heads)




#### 4. Connect the power wire , Turn on the power

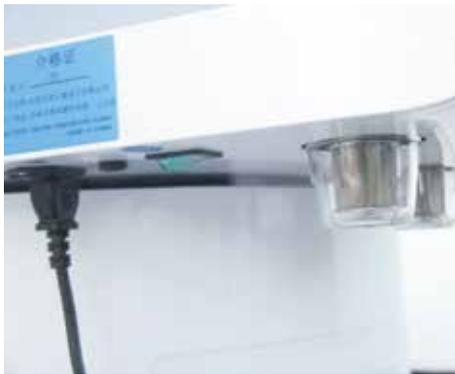

#### 5. Connect the pipe to the Mainframe



#### 6. Connect the drainpipe to the mainframe, Add the water

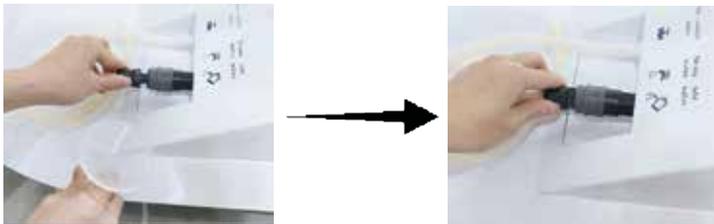


#### 7. Install the filter



Remark: Before use the machine , Pleast fullfill the water .As the machine starts to work , need to remove the water pipe

#### 8. Install the handle shelf

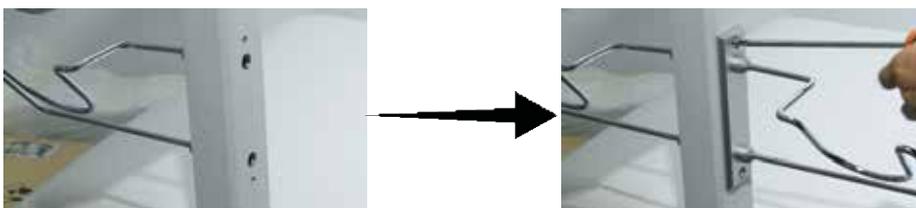

#### 9. Install the Holder




# Operation Instruction


## 1.Homepage operation instruction



## 2.Magic curve operation instruction

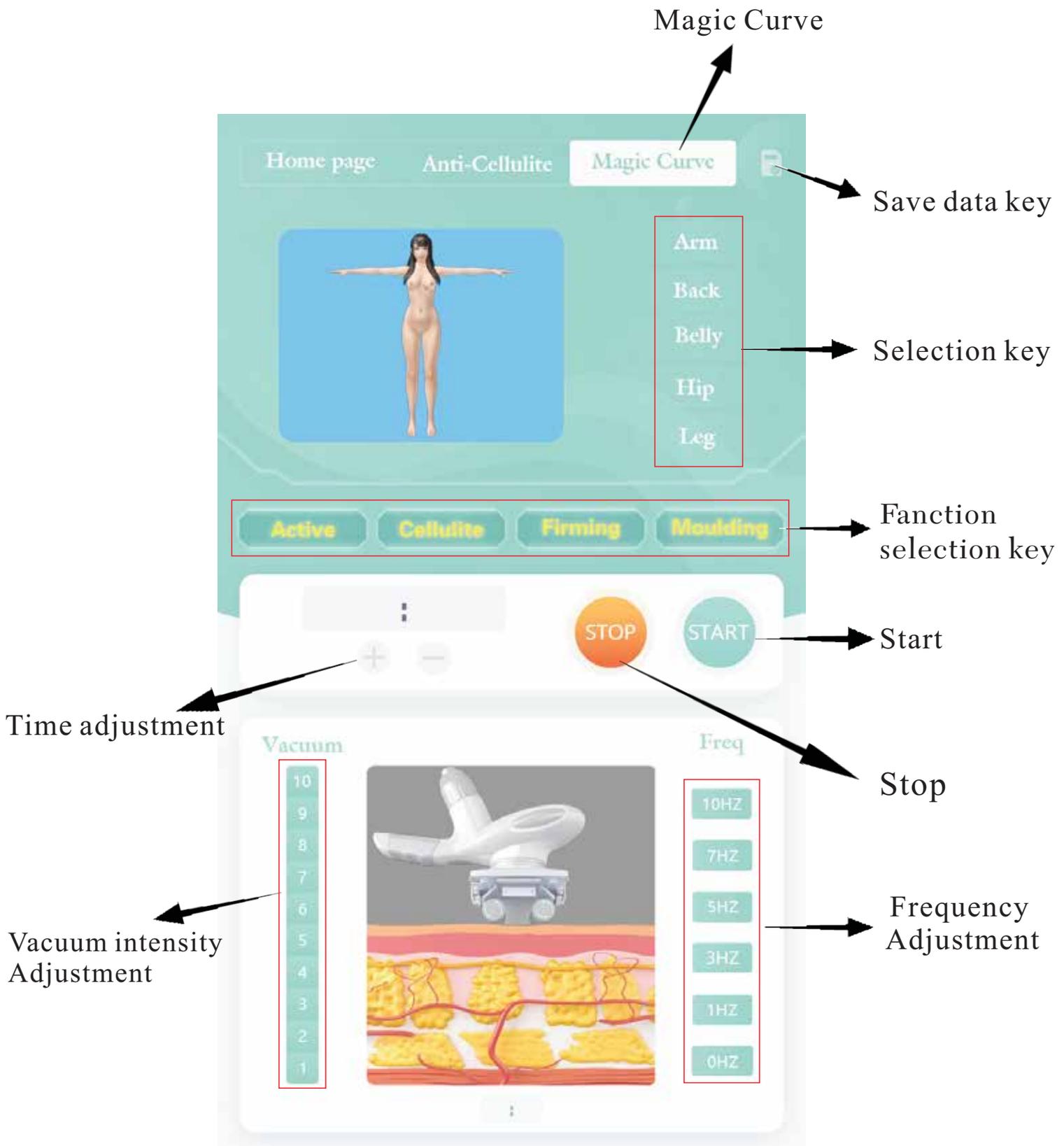


### 3. Anti-Cellulite Panel instruction

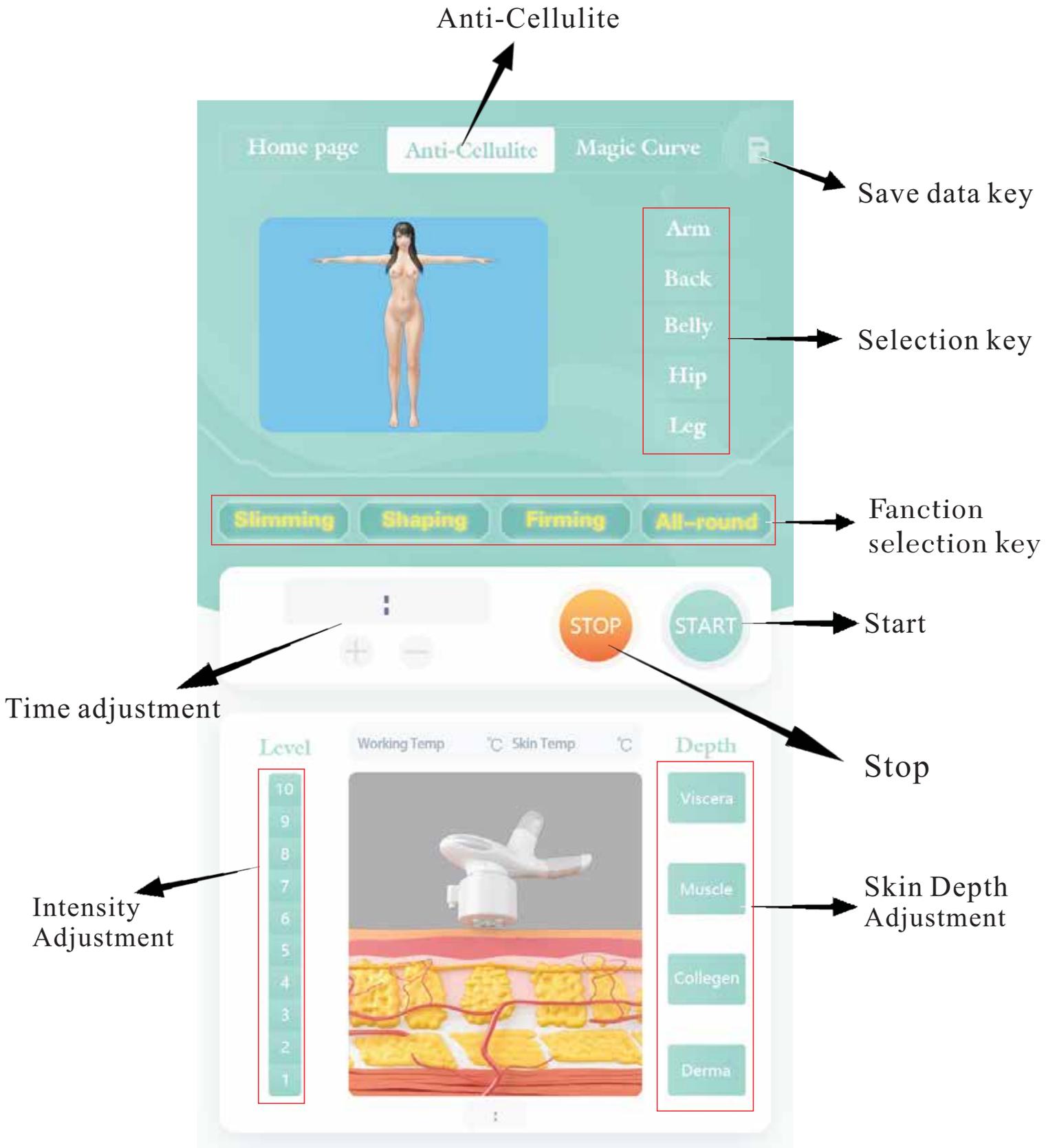

# Operation Instruction

## 1. Home Page Operation

(1) Connect the power wire , start the Power ,  
The screen display the Homepage

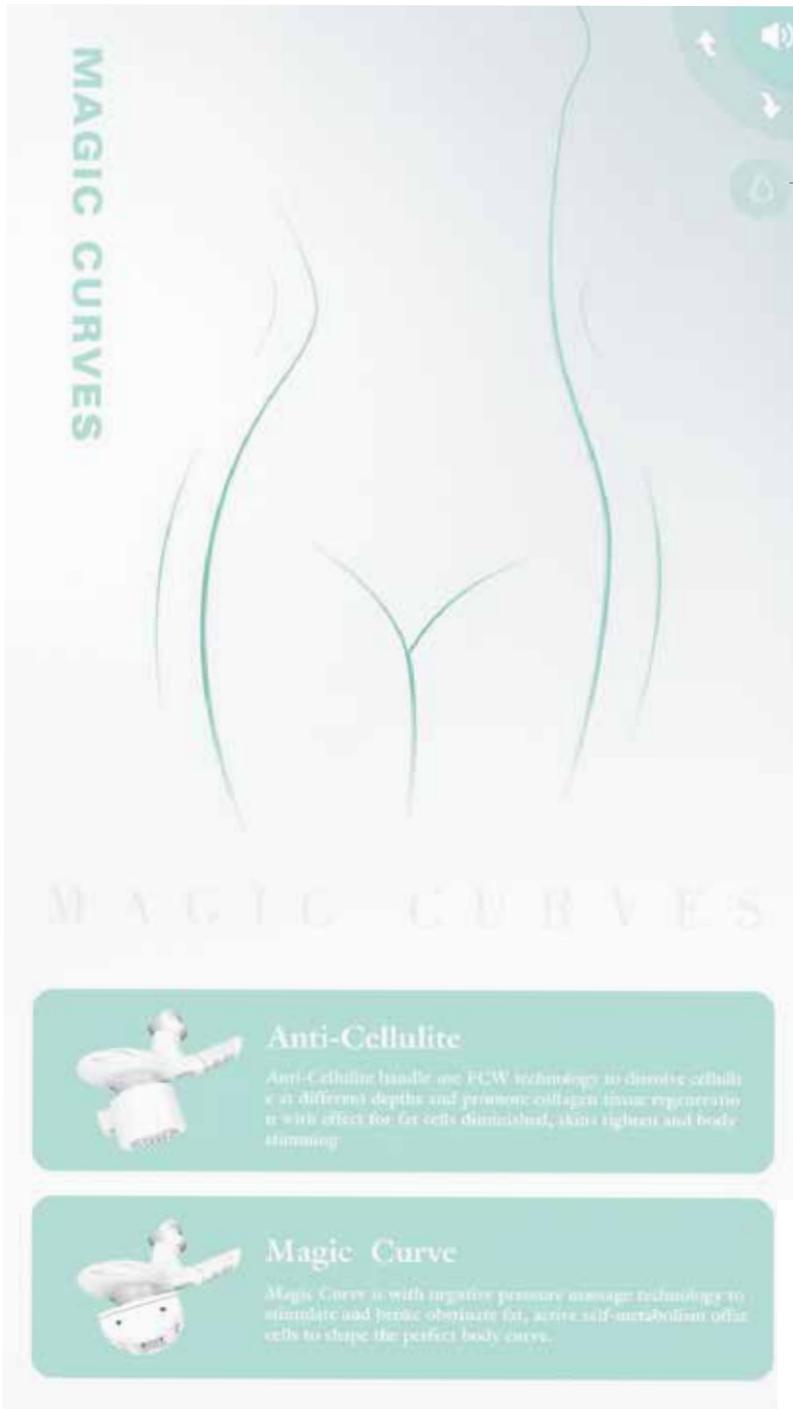

on/off voice

The bed up/down  
adjustion

Drain the useless water , After  
connect the drain pipe , press the  
homepage ,Pop-up window, select  
"Yes".

## 2. Magic Curve Operation instruction

(1) Press the Homepage, select the Magic Curve, Enter the operation interface, adjust the parameter

Step 1: Select the Part (Arm/Back/Abdomen/Buttock/Leg)

Step 2: Select the Function (Active/Cellulite/Firming/Moulding)

Step 3: Adjust the Vacuum Intensity (According to different customer)

Step 4: Adjust the Frequency intensity

Step 5: Setting the time


(2) Remark: The Magic Curve have 4 Modes: Active, Cellulite, Firming, Moulding



(3) Setting the Mode:

Four modes used for different Parts			
Active	Cellulite	Firming	Moulding
 <p>2 kinds of modes : The roller turn to left or turn to right</p>	 <p>1 Kind of Mode: The Roller Turn inward</p>	 <p>1 Kind of Mode: The Roller Turn outward</p>	 <p>2 kinds of modes : The roller turn to left or turn to right</p>
Function: Relaxing, Dredge lymph	Function: Celluliting, Lose weight	Function: Firming the Muscle	Function : Stimulate collagen synthesis. Sculptural body curve

(4) After setting the parameter, if you want to save the data, Press the top right icon, pop-up window, select "Yes", Then save the data

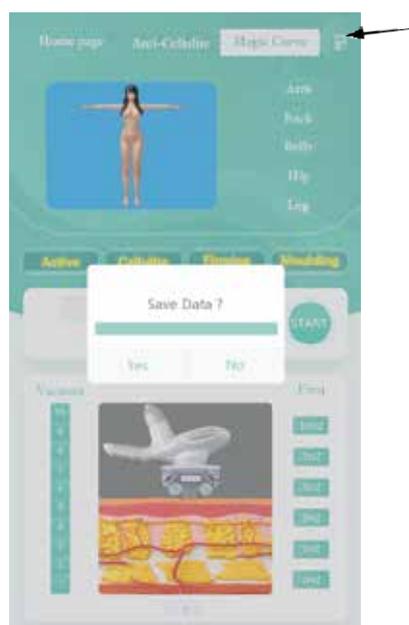


(5) After setting, Press the "Start", begin to work, If need to revise the parameter, Press the "Stop", Then revise it

### 3. Anti-Cellulite operation instruction

(1) Press the Homepage, select the Anti-cellulite, Enter the operation interface ,Setting the parameter


- Step 1: Select the Part (Arm/Back/Abdomen/Buttock/Leg)
- Step 2: Select the Function (Slimming/Shape/Firming/All-Round)
- Step 3: Adjust the Level (According to different customer)
- Step 4: Adjust the Depth

(2) Adjust the Temperature on The Anti-Cellulite Handle

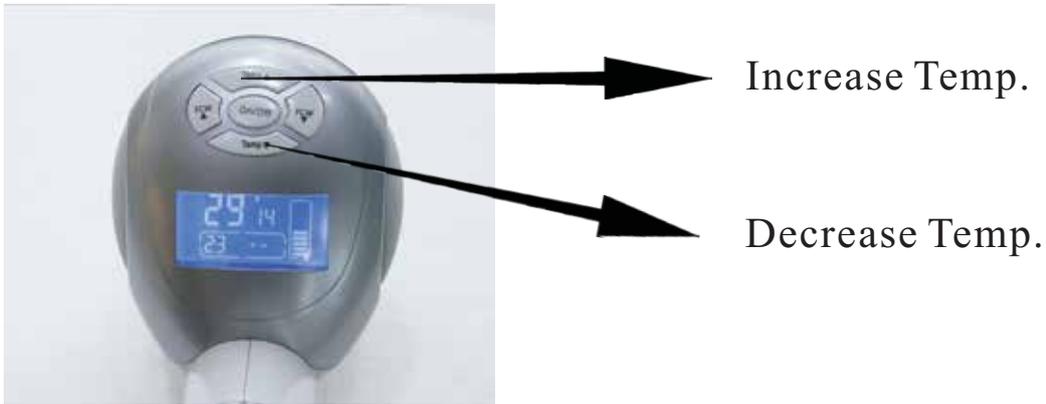

Remark: When the temp display "--", It means Stop cooling ,  
Normally No need cooling .

(3) After setting the parameter , if you want to save the date , Press the top right icon, pop-up window, select "Yes", Then save the date

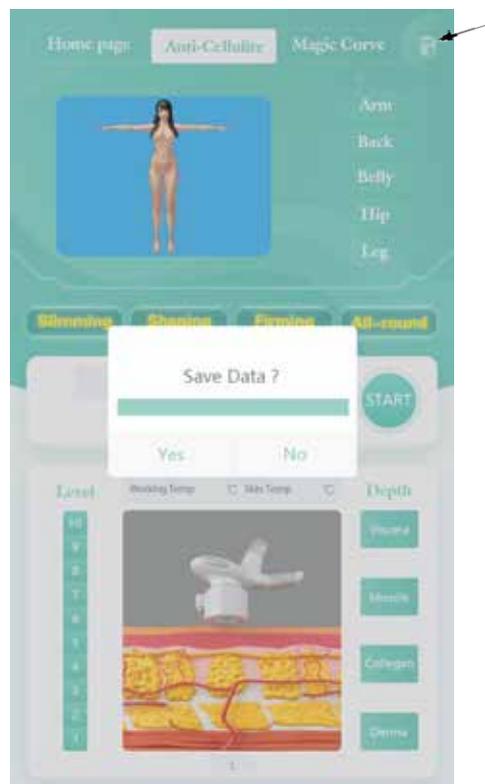

(4) After setting , Press the "Start", begin to work , If need to revise the parameter, Press the "Stop", Then revise it

# Solution with simple Malfunction

Failure	Solution
Power on, the machine no reponse	Check whether the power wire is in good connection with mainframe
	Check whether the power switch is on
Small suction no suction	Inspect whether the air line connected well with the main frame whether the main frame displays good
	Inspect whether the filter has not been cleaned for a long time or the filter screen is blocked Whether the inflator is broken or air leakage
During the operation ,the machine is overheat	Using for too long time, turn it off and let it naturally cool down
	Check whether the power is OK (inspected by technician)
No power	Check the fuse, if it is broken, replace it

## Specification

MEAS : 135\*62\*74CM

Power:800W

Voltage : 220V 50Hz

Electric current : 10A

# Packing list

ITEM:NW-8308				
Name	Unit	QTY	MEAS	Remark
Main	set	1		
Magic curve head	pc	2	One smooth head, One Treaded heand	
Magic curve handle	pc	1		
Anti-cellulite Handle	pc	1		
Massage cloth	Set	10		
Power wire	strip	1		
Seal	pc	2		
Spanner	pc	1		
Add water funnel	pc	1		
Accesstories Shelf (with display screen )	pc	1		
Filter	pc	2		
Fuse	pc	4		
Instruction	pc	1		
Accessotory box	pc	1		
Silicon Pipe	strip	1		

# Warm Tips:

## 1. Some matters need attention when Use Magic Curve handle

Before use the machine, please read carefully the following information:

- ♥If skin have rebefaction or have scar while using the machine , please stop operation immediately! If feel pain while using , please tell the operator reduce the suction in time .
- ♥When operation the machine , Do not keep the handle on the same part for a long time , Remove the handle on body or on face slowly .
- ♥The children which under 7 years old are forbidden .If the Children over 7 years old need use the machine , Need doctor's advice
- ♥People who Have a fever,Infection ,Inflammation,skin disease (sAcne,Eczema,Herpes,HPV),Fresh Scab,Serious or chronic disease (Cancer,AIDS,Blood disease )are Forbidden.
- ♥If you have a risk of Phlebitis, and had vein surgery within 2 months , Prohibition of use the machine .If have any questions,please make advice from your doctor
- ♥Pregnant women are forbidden.
- ♥After do surgery,please make a evaluate by your doctor if you can use the machine or not (We suggest use the machine after 3-6 months).Can use the machine after 3 months if eutocia ,after 6 months if cesarean.
- ♥The machine can use for any health body part expect the Eyes ,Throat,Ears ,glottis and genital.
- ♥Unless directed by a doctor, Please do not use the machine on ears ,Inside the mouth , neither to mucous membrane,nipples ,nevus.
- ♥The epileptic,Psychopath are forbidden .  
Remark:Please wear the massage cloth while using the magic curve

## 2. While using the Anti-cellulite, if there are the following cases , please do not use the machine

- ♥If have cardiac pacemaker,Internal defibrillator, Forbidden
- ♥If the operation body part have metal implant
- ♥People who have skin cancer , or before cancer lesions
- ♥People who have serious blood pressure
- ♥People who during pregnancy or sucking period
- ♥The operation part have bacteria or virus infection or opened wound
- ♥The operation part have the permanent filler . or had Fat filled ,Plastic surgery within 3 months

## 3. Notes for use the machine

- ♥Use the machine regularly, will make your body more health , Please do not stop use for a long time .If use the machine not following the right rules and precautions , The effecton will may not good as you want .If there is no effect after use the machine for several times ,or have any questions while using the machine , Please consult your doctor .There are some differenteffects because of the age ,living habit (sport style , late sleeping ,diet ,take antibiotics ), ect;
- ♥People still can use the machine if cure from the plastic surgery . But please consult the doctor

The machine only operated by professional

# Operation Cautions

- (1) The machine only for beauty salon , not for home use .The operator should be professional,The Non-profess operate will give rise to unnecessary hurt (The children andthe mentally disabled people are forbidden to operate )
- (2) Please do not disassemble the machine without our technician's guide or without the user manual giude
- (3) The machine should keep away from the water.when install the machine , please keep your hands dry
- (4) Please contact with the technician if the following occurs:
  - ▶ :The liquid splashed on the instrument or get wet in the rain
  - ▶ :The instrument have peculiar smell, Smoking or unnormal voice
  - ▶ :The power wire broken
  - ▶ :The instrument fall down or casing damage
- (5) Please do not put anything on the power wire , please keep away from the place which power wire will by stepping on
- (6) For safety , please turn off the instrument after use
- (7) Please do not insert anything others in the port , otherwise it may make fire or electric shock.If there are something fall inside of the instrument and you can't take them out, Please turn off the instrument , and contact with your local distributor or contact with us .
- (8) Please don't put the machine on the unstable trolley,Floor,shelf or car ,f fall down,the machine will be in serious broken.
- (9) To extent the instrument life ,please Turn on/off intercal time more than 1 minute
- (10) The voltage should in step with the regulation,otherwise the machine will may not work even broken the machine .
- (11) If the power wire broken , please contact with the seller
- (12) Our company reserves the right to modify the information in this introduction before bnotify! And keep the explanations right for the above.

