

**Ultrasonic Cavitation Vacuum RF Body**

**Slimming Skin Lifting Beauty Machine**

**MODEL: MS-54D1**



Dear users:

Thanks for choosing our company's product, in order to fully make use of the products, we sincerely suggest follow:

- ◆◆ Please read the instruction in detail and keep it well for further reading and consulting.
- ◆◆ Please follow the instruction to fit and operate the machine correctly.
- ◆◆ Please do not remove and change any accessories of the machine.
- ◆◆ It forbids others to open and remove the mainframe except the authorized people in the company.
- ◆◆ It forbids others to change and maintain the machine except the professional people.
- ◆◆ Please contact us if there is any trouble during operation.

After Sale service

◆ We offer ONE YEAR warranty. Within warranty period, all exchanges or replacement due to defects are free. (Buyer is responsible for shipping charges).

If the machine is over warranty, please also do not hesitate to contact us. we will assure you of our best service all the time.

This product does not belong to medical, please don't use for disease treatment!

### **FEATURES:**

1. Painless treatment focus RF energy at the correct position, compared to other radio frequency technology. It uses low energy and high frequency, safe and effective.

2. Aim at the skin surface and deep location, using a complex method to control the different currents and energy, direct access to the different skin layer. Does not cause the phenomenon of uneven skin.
3. Selectively targeting fatty tissue, to avoid heating other fat, to achieve the fastest treatment effect.
4. The entire process is completed without the need of surgery and anesthesia.
5. With the most useful 40KHZ ultrasonic cavitation system.
6. Without side effects and the risk of gaining the weight back, the results are rather obvious. Does not affect the normal work and life.

### **FUNCTIONS:**

Firming the arms, legs, thighs, buttocks, waist, back, abdominal muscles and reshape the body figure Improve and treat the orange peel like fatty tissues of different serious levels With the effects of soothing, decompression and pain relief (such as joint pain, sciatica), promote the regular mechanical aerobic exercise of skin tissue, rapidly relieve stress and remove fatigue. Firming, lifting and compact the skin and mould the face shape circulate and activate cells, promote metabolism and so that 100% more collagen and elastic fibers may be gained for improving the facial and body skin.

### **OPERATION PROCEDURES:**

40Khz Ultrasonic handle-strong sound wave explosion fat head

Press "TIME SET" to adjust the time, clicking "-" to decrease the strength or click "+"



to increase it. Press  to start/Stop the strong sound wave explosion fat head.

Model A is with continuous wave, which is used for the area with thin fat;

Model B is with the dis-continuous wave, which is mostly used for the area with thick fat.

Operate with the 40KHZ cavitation head for 15-20 minutes with mainly the spiral movement on body.

### **Vacuum Bipolar RF Handle**



Panel: Press the image  in the display screen to select Vacuum bipolar

RF function, select the suitable RF intensity(adjust according to the customer's acceptable level, 3-6 is ok.), suction time and release time, then press the START/PAUSE button, then the vacuum bipolar rf probe starts to work.

Beautifying: Use the probe to massage the specific part of skin circularly after applying the sliming cream for about 30 to 40 minutes. It could quicken the blood circulation in the part of skin treated and has the effect of decomposing fat, etc.

Matters needing attention:

1. Apply sliming cream or cooling gel to the part of skin to be treated before the treatment.
2. Keep close contact between the probe and the skin.
3. Adjust the intensity from low level to high level.
4. Please do not irradiate eyeball directly with colored light.
- 5 Operate with the bipolar vacuum head for 15-20 minutes with mainly the spiral movement on body.

### **Multipolar RF Head for Body**



Press the image in the display screen to select the Multipolar RF function.

Press "TIME SET" to adjust the time, clicking "+" to increase the RF intensity or "-" to

decrease RF intensity, press "work" to start the operation of negative pressure vacuum RF head. Apply gel to the negative pressure vacuum RF head, point it to the position for losing weight and operate in pulling technique(drag fat at the position for losing weight to the position where it is closer to the lymph gland) for about 20 minutes.

Notes:

1. Spread gel or essence on the treated positions for hairdressing evenly.
2. In use, start with low-energy. We suggest to try for the proper energy level on the back of clients' hands before using on body of them.
3. Do not use essential oil as the medium.
4. In operation, start pulling only after negative pressure head is attached to skin.
5. Turn off the instrument probe when it is not in use.
- 6 Operate with the tripolar RF head for 15-20 minutes with mainly the spiral movement on body.

### **Quadrupole RF Head for Face and Body**



Press the image in the display screen to select the Quadrupole RF function.

Press "TIME SET" to adjust the time, clicking "+" to increase the RF intensity or "-" to decrease RF intensity, select the suitable intensity (adjust according to the customer's acceptable level, 3-6 is ok.), and at last press "work" key. The face RF starts to work.

Beautifying: Use the probe to massage the specific part of skin circularly after applying the sliming cream for about 30 to 40 minutes. It could quicken the blood circulation in the part of skin treated and has the effect of decomposing fat, etc.

Matters needing attention:

1. Apply sliming cream or cooling gel to the part of skin to be treated before the treatment.
2. Keep close contact between the probe and the skin.
3. Adjust the intensity from low level to high level.
4. Please do not irradiate eyeball directly with colored light.
5. Operate with the tripolar RF head for 15-20 minutes with mainly the spiral movement on face.

### **Tripolar RF Head for Face and Eyes' Around**



Press the image in the display screen to select the Tripolar RF function.

Press "TIME SET" to adjust the time, clicking "+" to increase the RF intensity or "-" to decrease RF intensity, select the suitable intensity (adjust according to the customer's acceptable level, 3-6 is ok.), and at last press "work" key. The face RF starts to work.

Spread gel or wrinkle smoothing essence on the treated positions for hairdressing (generally 20-30G each time), operate with the facial RF head for 15-20 minutes with mainly the spiral movement on face and eye around.

**Cold probe** for calming down the skin and shrinking pores.

Press the red on/off button on the probe to start/pause the probe, operate 5-10 minutes.

#### **PACKAGE INCLUDING:**







- Multipolar RF Head for Body x 1
- 40KHz Cavitation Head x 1
- Vacuum Bipolar RF Head x 1
- Tripolar RF Head for Face and Eyes' Around x 1
- Quadrupole RF Head for Face and Body x 1
- Power cord x 1

### **Instrument Function and Treatment Principles**

#### **1. 40Khz Cavitation**

Operation:









1. Press  and  to adjust the working time.
2. press  to decrease the energy , and press  to increase the energy.
3. Press  to select model, two models can choose(A, B)
4. Press  to start work or pause operation

## 2, Tripolar rf operation:

Operation:



1. Press  and  to adjust the working time.
2. press  to decrease the energy , and press  to increase the energy.
3. Press  to select model, two models can choose(A, B)
4. Press  to stark work or pause operation

Beautifying: Use the probe to massage the specific part of skin circularly after applying the sliming cream for about 30 to 40 minutes. It could quicken the blood circulation in the part of skin treated and has the effect of decomposing fat, etc.







Notes:

1. Apply sliming cream or cooling gel to the part of skin to be treated before the treatment.
2. Keep close contact between the probe and the skin.
3. Adjust the intensity from low level to high level.
4. Please do not irradiate eyeball directly with colored light.

### 3. Quadrupolar RF for face

Operation:



1. Press  and  to adjust the working time.
2. press  to decrease the energy , and press  to increase the energy.
3. Press  to select model, two models can choose(A, B)
4. Press  to stark work or pause operation







#### 4.Sextupolar Body RF handle

Being integrated with the most advanced RF technology and radio frequency, the instrument may directly reach the deep-seated fat body and have the excellence of targeted positioning RF. In the fast-active state, fat cell tissue may generate friction heat, increase local temperature and remove excess fat and toxin from the body through sweat gland, enter hepatic circulation and lymph and ultimately achieve the effect of dissolving fat.

With controlled depth technology, inlaid diamond particles of different sizes may rubs kin along its pattern, adjust suction strength by means of vacuum draw and rub withne gative pressure strong force to directly explode thick fat. Thus, the effects are veryobvious: Enhance blood circulation, accelerate the decomposition of fat, take away wastes, enhance the effect of removing cellulite, effectively improve the state of areola tissue, remove stubborn fat and improve the sufficient absorption condition of diet products.













1. Press  and  to adjust the working time.
2. press  to decrease the energy , and press  to increase the energy.
3. Press  to select model, two models can choose(A, B)
4. Press  to stark work or pause operation

## 5.Vacuum rf head

Operation:



11. Press  and  to adjust the working time.
2. press  to decrease the energy , and press  to increase the energy.
3. Press  to adjust the suction, press  to adjust the release, usually suction must higher than release.
4. Press  to select model, two models can choose(A, B)
5. Press  to stark work or pause operation.


## Notice:

1,Please insert the filter to the back of the machine at your first use. Or the vacuum bipolar RF probe wont have suction.



2,please use gel before you use the RF probes, and make sure the probes completed touched with skin, or you will not feel the heat

3,If the vacuum suction is weak, please adjust the following knob:

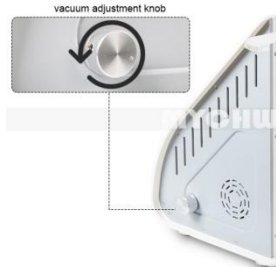

**4,If the vacuum probe still dosent have suction after inserting the filter,adjusting the suction,release time,adjusting the vacuum knob, please hear the vacuum pump is working or not. If the vacuum pump is working, please take off the vacuum probe from the machine, then use your finger to touch the vacuum connector on the machine to see whether you can feel the suction( when you test, please make sure the vacuum function is on). If you can feel the suction on the vacuum connector on the machine, it shows it will be the problem of the vacuum probe, then just need to exchange one new vacuum probe;  
If you cant feel the suction, please take one video for your operating and testing, then send the person who sale you the machine. Then we solve the problem very fast.**

**5,If your machine has some problems, or you have some doubts for the machine urgently, please contact +86 18948755929 (whatsapp, Wechat) directly.**

## BUTTOCKS

### ILLUSTRATION of OPERATION

## FACE

Treated time: 30 minutes

1. Massage the mandible center in circles to produce deep heat.

2. Massage lower jaw in lines.

3. Massage the triangle zone of both sides of the face in circles.

4. Massage from jaw to angulus oris to ear in lines.



## THIGH

Treated time: 30 minutes

1. From down to upper, pushing to the groin to dredge the lymph.



2. From down to upper, circling push by anticlockwise can help decomposing fatness.



3. Also can push by come-and-go to decompose fatness.



4. Pull from the knee and muscle texture to upper, can improve the curve.

Treated time: 20 to 30 minutes

1. Along the lymph direction, pulling to the waist.

2. From upper to down, by anticlockwise gesture, pull come-and-go.

3. Along the muscle of arm, pull up to lift and tight the muscle.

## BACK



Treated time: 20 to 30 minutes

1. First by come-and-go, pushing the bladder nerve 2 to 3 times.

2. Circling by anticlockwise to stimulate the underarm lymph node.

3. On the back, by anticlockwise circling, can help to decompose the fatness.

4. Using lymph drainage gesture, pull the toxin to the lymph node, can help improving the back curve.

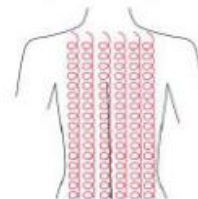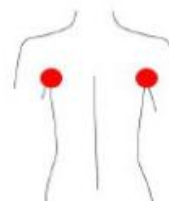

## ARM

Treated time: 30 minutes


### BACK SIDE

1. From the inner elbow to the armpit, doing lymph drainage.

2. Circling to stimulate the lymph node.

3. By anticlockwise or come-and-go pushing, decomposing the fatness.

4. From the elbow to the outer, doing tightening gesture.

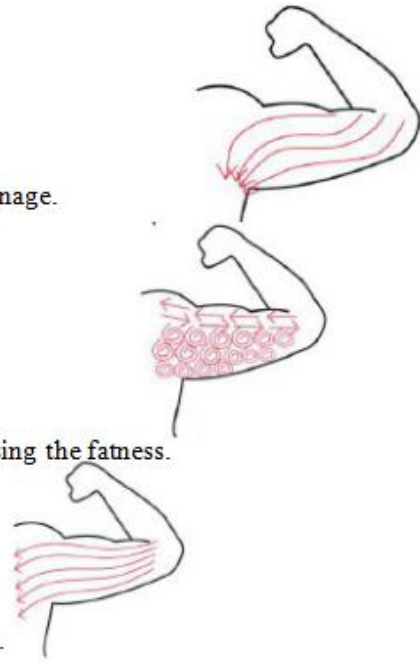


## Facade

1. From the elbow to the outer, doing tightening gesture.


## Abdomen

Treated time: 20 to 30 minutes


1. From small to large, by clockwise direction, circling around the navel, can help peristalsis of the large intestine.

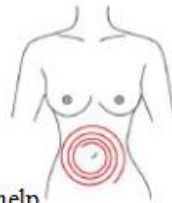


2. From small to large, by anticlockwise direction, circling push down.

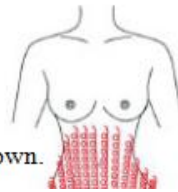

3. From the belly and muscle texture, pull to the groin.

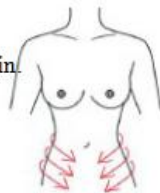

4. Through lymph drainage gesture, taking the toxin to the place of groin.

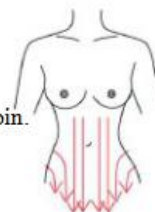

Attention:

1. Pregnant women or women during in menses.
2. Epileptic.
3. Patients with malignancy.
4. Patient whose wound after operation has not healed up.
5. Acute inflammation or epidemic patients.
6. Whom with heart diseases or with heart pacemaker.
7. Who with kidney (gall-stone) disease.
8. Who was embedded metal object or silica gel.
9. Who in menses, birth control period, emission incontinence period, or accepting the belly operation.
10. Whose body always takes much inner hot.
11. Who has the genetic hypersensitivity.

Notice:

1. Be sure to use the special ultrasonic gel.
2. Avoid knocking the us head.
3. For avoiding burning the head, during operation, please prepare enough ultrasonic gel.
4. Don't stay in one place, avoid treating on the bone.
5. Don't use disinfectant product to disinfect the us head. We suggest the wet cotton or dry towel enough.
6. Check the machine power cord if it is connected well.
  
7. If the machine will not use for a long time, please switch off it, and power off.
8. Please take off all metal decoration from the operator and patient's body.
9. If continuous using 1 hour, please pause the machine about 10 minutes, then using again.
10. During the operation, be sure not to accept other treatment.